



HIPAA COVER INSTALLATION

1. Lay the cover on the floor with one of the zipper covers facing up.



2. Unzip one cover and flip it up.




3. Move the zippered strip towards you, exposing the two cart belts.



4. Move the cart over the strip as illustrated.



5. List the cover up and over the cart.

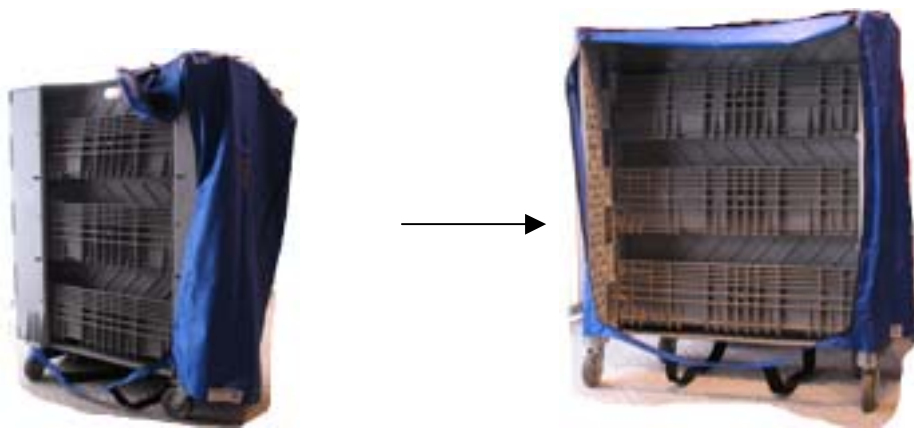

Open the cart doors and lower the shelves into place, stretching the cover over the upper corners of the doors. Zip closed the cover and open the second side zipper.

Open the doors and lower the shelves into place. *NOTE- the cover will be tight but it will fit over the corners of the doors.



Once you have the cover on the cart and the shelves are down, you can zip up the cover and Congratulations!



To remove the cover you must close the shelves and doors.